

MD[®] Pen

Instructions for use



Step 1

Turn the base of the MD Pen counter clockwise into the OPEN position. Make sure to hear the click.



Step 2

Press the base to dispense on localized area.




Step 3

Apply the preparation using the soft silicone tip by rubbing it into the area of application as directed.



Step 4

Turn the base of the MD Pen clockwise into the CLOSE position. Make sure to hear the click.

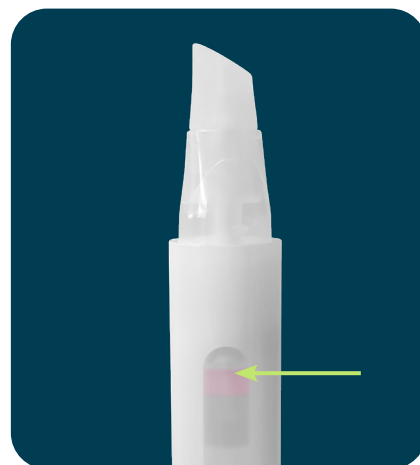

Step 5

Call for replacement, once the piston nears the domed part of the view window.

For more information

USA 1-800-932-1039 medisca.com
CAN 1-800-665-6334 medisca.ca

Copyright© 2024 Medisca Pharmaceutique Inc. All rights reserved.