

# MD<sup>®</sup> Syringe

## Instructions for use



### Step 1

Turn the base of the MD Syringe counter clockwise into the OPEN position. Make sure to hear the click.



### Step 2

Press the base to dispense on localized area.



### Step 3

Apply the preparation using the soft silicone tip by rubbing it into the area of application as directed.



### Step 4

Turn the base of the MD Syringe clockwise into the CLOSE position. Make sure to hear the click.



### Step 5

Call for replacement, once the piston nears the domed part of the view window.

### For more information

USA 1-800-932-1039 [medisca.com](http://medisca.com)  
CAN 1-800-665-6334 [medisca.ca](http://medisca.ca)

Copyright© 2024 Medisca Pharmaceutique Inc. All rights reserved.