


8790-01

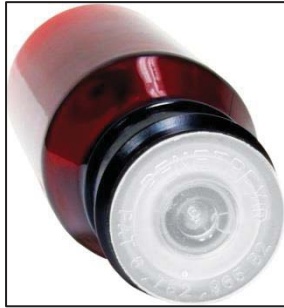

SealSafe Bottle Closure, Plastic, Natural, 24 mm, pk/100 (ANDWIN SCIENTIFIC)

Note: Bottle not included.

The Patented SealSafe Bottle Closures allow oral syringes to draw the exact amount needed while reducing the risk of spill, evaporation and contamination. They fit snugly into most bottles in their size range allowing original child-resistant (C/R) caps to fit over the closures. SealSafe Closures feature flexible sealing rings for compatibility with a range of bottle sizes. To use, simply insert the tip of an oral syringe into a SealSafe Closure, draw a measured dosage and remove the tip. The closure quickly recloses providing a safe and clean seal.

Features:

- ✔ Maximize drug dosing accuracy
- ✔ Prevent evaporation, spills & contamination
- ✔ Original child-resistant and standard caps fit over the adapters
- ✔ Easy-to-read syringe dose marks never enter the bottle
- ✔ Prevent waste by always getting the last drop of medication
- ✔ Limit accidental access even when C/R caps are removed

The above information is a direct transcription of information provided to MEDISCA® from the Specification Sheet and/or Certificate of Conformity provided by the manufacturer/supplier.

Operating Instructions:



1. Select a bottle with a compatible neck size.



2. Remove the screw-cap.



3. Press a SealSafe™ Adapter into the neck of the bottle.



4. Insert the tip of an oral syringe into the seal, turn the bottle upside-down and draw a desired dosage.



5. Turn the bottle right-side-up and remove the syringe tip. The SealSafe™ Adapter reseals after the tip is removed. Replace the cap onto the bottle.

 SealSafe™ Adapters should be tested for use with specific containers, liquids and oral syringe tips to confirm fit and compatibility and to ensure optimal performance.

The above information is a direct transcription of information provided to MEDISCA[®] from the Specification Sheet and/or Certificate of Conformity provided by the manufacturer/supplier.