



STEP 1:
Screw Luer Syringe to the Luer lock tip of the Micro.

NOTE: Requires no extra Luer Lock adaptor.

STEP 2:
Select Applicator tip and press on by pushing straight down (important) against a hard surface.

NOTE: For the Soft Angled Applicator placement requires tip to be aligned where lower slant of tip is on the same side as Refill Badge of the Micro tube.

STEP 3:
The actuator bar should seat completely to the plunger before priming. Advance the actuator to engage plunger by manually turning/CLICKing base until contents are dispensed, then wipe off. Be sure to align Micro base with the triad tube.

STEP 4:
Screw on cap securely to protect tip and prevent evaporation.