



PRODUCT NO.: 8679



TOPI-CLICK APPLICATOR

35mL, 0.5mL per 2 Clicks, Plastic, White

DESCRIPTION: Topi-Click is a metered cream/gel dispenser and applicator for topical compounds and is easy to use for all types of patients. Topi-Click is easier to use and fill than oral dosing syringe.

FEATURES:

- Domed applicator pad for hands-free application – hand absorption can affect treatment outcome by reducing dose absorbed by prescribed treatment area
- Audible, visual and tactile click makes it easy for all patients to accurately dose medications
- “Refill” indication for prompting on-time re-buys that help eliminate missed applications
- Lower cost with simplified labeling – labels as fast and as easily as a prescription bottle
- Easier to fill and use with increased control over plunger movement than a topical syringe
- Now equipped with a flip tab for easier leverage to pop the top
- Friendly reminders of instructions on turn-dial base to help patients
- Applicable for total dispense weights of approximately 5g to 35g of topical preparations
- Turn the base “1 click” (a ¼ turn) to dispense one metered amount; doubling or tripling the dose amount is as simple as turning to the 2nd or 3rd click
- Made of FDA medical-grade polypropylene material
- UV block plastic protects the contents from UV rays


ADDITIONAL INFORMATION:

Applicable Products:

- 7947: Topi-Click Filling Station
- 7785: Modified 1L Unguator Jar
- 7778: Topi-Click Filling Nozzles


User Instructions:

Step 1




Pop off the cap of the Topi-Click® container by placing your thumb under the tab on the protective outer cap and pressing upward.

Step 2



Turn the base to dispense the number of clicks specified on label.




When rotating base, **watch** the Visual Indicators

Indicator Node on Body is fixed at the Click Point

Indicator Column moves with Base. This shows Node in line with column.


LEAVE IN THIS POSITION AFTER EVERY CLICK
(The Home Position)

Step 3




Apply treatment by rubbing the cream, which is now on the top of the applicator pad, to the area of skin as directed.

Step 4



Prevent evaporation of next dose, by snapping the cap on completely.

Step 5



The **bottom** of the plunger is the indicating line; when it reaches the embossed line **above** the word "Refill," there are approximately 12 clicks left. Approx. 32 clicks remain at the line **below** the word "Refill" as shown above.

Page 2 of 2

🇺🇸 1.800.932.1039 | 🇨🇦 1.800.665.6334 | 🇩🇪 1300 786 392 | WWW.MEDISCA.COM

The above information is a direct transcription of information provided to MEDISCA® from the Specification Sheet and/or Certificate of Conformity provided by the manufacturer/supplier.