



**MEDISCA**<sup>®</sup>  
YOUR TRUSTED PARTNER IN COMPOUNDING

**PRODUCT NO.: 5665**



## **Applicator, Topi-Click Micro, 9mL, 0.05mL\Click, White.**

**Non-Sterile**

**Description:** Topi-Click is a metered cream/gel dispenser and applicator for topical compounds and is easy to use for all types of patients. Topi-Click is easier to use and fill than oral dosing syringe

### **FEATURES:**

- ✔ Automates, metered topical dosing that helps providing an accurate and consistent dosage every time.
- ✔ Audible, tactile and visual clicks indicate measure of dose (One CLICK = 0.05 mL).
- ✔ Eliminate the need to manually mark the dose.
- ✔ Innovative triad-shape allows for precision handling
- ✔ Built-in REFILL Badge helps patients reorder medication on time.
- ✔ Applicator reduces the risk of hand transference to others.
- ✔ Offers applicator selections for customized dosing.
- ✔ Eliminates user-variability.
- ✔ Compatible with EMP Jar Quick-Fill Adaptor Filling
- ✔ Compatible with Luer-Lock Syringes.
- ✔ Turn the base “1 click” (120 degree rotation) to dispense one metered amount; doubling or tripling the dose amount is as simple as turning to the 2nd or 3rd click
- ✔ Made of FDA medical-grade polypropylene material
- ✔ UV block plastic protects the contents from UV rays.

### **TECHNICAL SPECIFICATIONS:**


Capacity (mL)	9.75
Material	Product Polyethylene
Dosing (per Click)	0.05mL per 120 degree rotation
Storage (°F\ °C)	35-160 \ 2-71

**USA | CANADA | AUSTRALIA PACIFIC | EUROPE**

[WWW.MEDISCA.COM](http://WWW.MEDISCA.COM)

Shelf-Life	5 years
Label Size ( W X H)	2.5'' x 2.75''
Approximate Residual (mL)	0.15
Approximate Priming (mL)	0.15ml or 3 clicks

## Additional Information

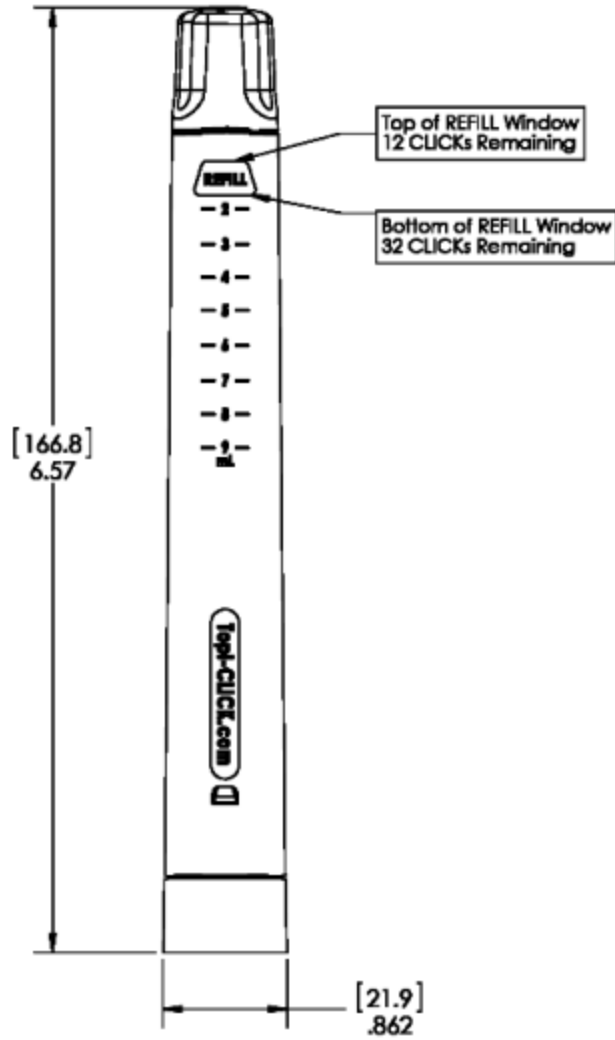

**USA | CANADA | AUSTRALIA PACIFIC | EUROPE**


[WWW.MEDISCA.COM](http://WWW.MEDISCA.COM)



**MEDISCA®**  
YOUR TRUSTED PARTNER IN COMPOUNDING


## User Instructions

**Step 1:** Unscrew the cap of the Topi-CLICK Micro® and set aside for recapping after use.

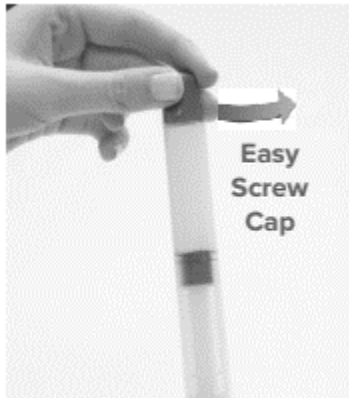

### Step 2:

Holding the Micro® in an upright position, turn/CLICK the Base the number of CLICKs or mLs specified on the prescription label. Use the Audible, Tactile and Visual CLICK points of the triad-shaped Base as dosing indicators. Be sure to align Base with the Triad Tube to ensure dose is fully dispensed.

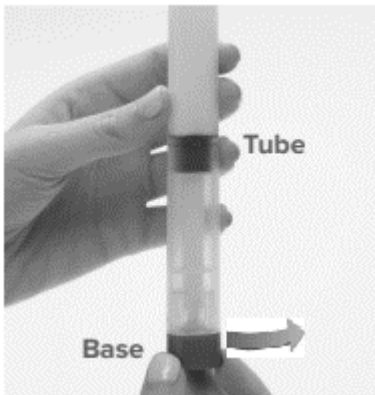

USA | CANADA | AUSTRALIA PACIFIC | EUROPE

WWW.MEDISCA.COM


**Step 3:**

Apply topical medication with built-in applicator as directed by physician or pharmacist. Continue rubbing until preparation is completely absorbed into the skin. **IMPORTANT:** The built-in applicator lessens the risk of transference to unintended areas. No need to use your fingertips to apply medication.



**Step 4:**

Screw on cap after each dose to prevent evaporation. Be sure cap is secured completely.

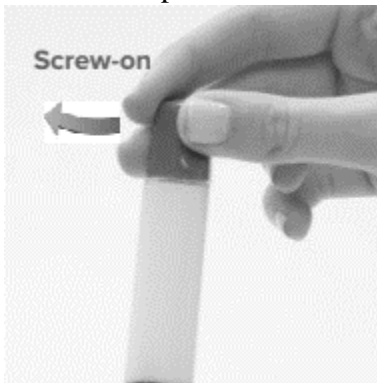

**USA | CANADA | AUSTRALIA PACIFIC | EUROPE**

[WWW.MEDISCA.COM](http://WWW.MEDISCA.COM)