



STEP 1:
Unscrew the cap of the Topi-CLICK Micro and set aside for recapping after use.



STEP 2:
Holding the Micro in an upright position, turn/CLICK the **Base** the number of CLICKS or mLs specified on the prescription label. Use the Audible, Tactile and Visual CLICK points of the triad-shaped **Base** as dosing indicators. Be sure to align **Base** with the **Triad Tube** to ensure close is fully dispensed.



STEP 3:
Apply topical medication with built-in applicator as directed by physician or pharmacist. **Continue rubbing until preparation is completely absorbed into the skin.**

NOTE: *The built-in applicator lessens the risk of transference to unintended areas. No need to use your fingers to apply medication.*



STEP 4:
Screw on cap after each dose to prevent evaporation. Be sure cap is secured completely.

When the **top** black line of the **Plunger** reaches the embossed line below the word “REFILL” approximately 32 clicks are left. Approximately 12 CLICKS remain at the line above the word “REFILL”.