



## Topi-CLICK PERL, Rose Quartz, Vaginal Dosing System w/4mL Vaginal Applicator , (35mL,0.25mL/Click)

**Description:** The elegantly designed and comfortable Topi-CLICK Perl is the revolutionary first vaginal dosing system and vaginal applicator design that is comfortable and accurate, yet easy to use. The Perl was designed with the patients, pharmacist and the physicians in mind.

### Features:

- Perl Snap-On Vaginal Applicator is made with clear, flexible soft medical grade polymer
- Automated Topi-CLICK dosing for accuracy and consistency through entire regimen
- Less cost, less clean-up, less waste
- Convenient, fast and easy to use
- Each click moves the plunger approximately 0.25mL of volume space, thereby dispensing 0.25mL of cream on average
- The most common number of clicks used for one dose is 2 clicks that moves the plunger 0.5mL volume space
- Filling with approximately 35mL, the Topi-CLICK will provide approximately 64 doses
- Can be labeled like a prescription vial – simplified labeling for lower cost – and easier to read
- Domed applicator pad for application without dispensing on to hands first
- Audible, visual and tactile click registration for user recognition
- A “Refill” reminder system helps patients to reorder in a timelier manner
- Designed to resist accidental discharge in purse, pocket or luggage

USA | Canada | Australia | International

[medisca.com](http://medisca.com)

- BPA and Phthalate free | UV block
- One time use only

### Technical Specifications:

|                      |                                                                            |
|----------------------|----------------------------------------------------------------------------|
| Dispenser Volume     | Up to 37 mL                                                                |
| Delivery Volume      | Up to 34 mL                                                                |
| Dispense per CLICK   | 0.25 mL                                                                    |
| Approximate Residual | 2.5 mL                                                                     |
| Approximate Priming  | 0.50 mL or 2 CLICKs                                                        |
| Doses Provided       | Approximately 32 mL or 64 doses (at 2 CLICKs per dose or 128 CLICKs total) |
| Height (mm)          | 99                                                                         |
| Width (mm)           | 46                                                                         |

USA | CANADA | AUSTRALIA PACIFIC | EUROPE  
[WWW.MEDISCA.COM](http://WWW.MEDISCA.COM)

Please note that the information contained herein is for informational purposes only. The information is based on recognized compendia, manufacturers' instructions and/or other authoritative sources and texts. MEDISCA takes no responsibility for the validity or accuracy of this information. Customer MUST refer to USP, FDA, Health Canada, TGA and/or their State Board to ensure they comply with the specific regulations and ensure that appropriate procedures are in place.